

MED⁹EL



SAMBA 2 GO

Quick Guide

AW52817_1.0 (English EU)


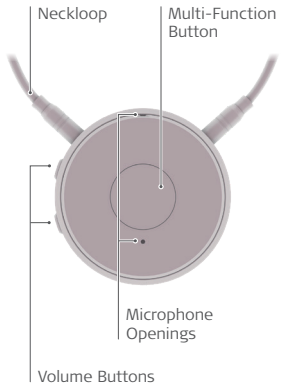


Table of Contents

SAMBA 2 GO	1
Wearing SAMBA 2 GO	2
Switching SAMBA 2 GO On/Off	3
Pairing with your Audio Processor	4
Adjusting Settings	6
Streaming with SAMBA 2 GO	7
Pairing with your Bluetooth Device	10
Making Phone Calls	12
Connecting via Cable	15
Using Remote Microphones	16
Charging SAMBA 2 GO	17

The SAMBA 2 GO Quick Guide is intended as an easy-to-use reference when handling your SAMBA 2 GO. For detailed information on the intended use, indications, contraindications, warnings and precautions of SAMBA 2 GO, please refer to the instructions for use.


SAMBA 2 GO

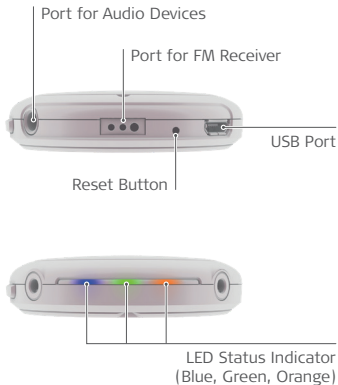

2 SAMBA 2 GO

Wearing SAMBA 2 GO

In order to function correctly, your SAMBA 2 GO must be worn with the neckloop around your neck. The neckloop is also the antenna.

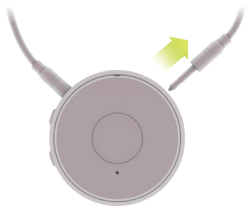

- 1 Unplug one neckloop plug and place the neckloop loosely around your neck.



- 2 Plug in both neckloop plugs all the way.

Switching SAMBA 2 GO On/Off



1 To switch on SAMBA 2 GO, press the multi-function button until the green LED turns on. This takes approximately 2 seconds.

2 To switch off SAMBA 2 GO, press the multi-function button until the green LED fades off.

Pairing with your Audio Processor

You only have to pair your audio processor with SAMBA 2 GO once. After that, the connection will be automatically established when both your audio processor and your SAMBA 2 GO are switched on. If you are a bilateral user, you can pair both audio processors at the same time.



- 1 Switch off your audio processor.
- 2 Make sure your SAMBA 2 GO is switched off. Place it around your neck.



- 3 Press the multi-function button on SAMBA 2 GO until the green LED turns on. The green LED will flash while searching for audio processors nearby.

- 4 Switch on the audio processor while the green LED is flashing.
- 5 Pairing is complete when the green LED stops flashing, is illuminated for 5 seconds, and then is no longer illuminated.



- ! During initial activation only, the blue LED will now flash to show SAMBA 2 GO is ready to be paired with a Bluetooth device.
- If you want to pair a Bluetooth device, skip to step 3 on page 11.
 - If you don't want to pair a Bluetooth device right now, press the multi-function button to stop the blue LED flashing.

Adjusting Settings

SAMBA 2 GO helps you to make adjustments of your audio processor settings, as pre-defined by your audiologist or healthcare professional.

- 1 To adjust the volume, press the upper or lower volume button once to change the volume step by step. Alternatively, press and hold the button until the desired volume is reached.
 - 2 To mute or unmute the microphones, press the upper and lower volume buttons together for approximately 2 seconds.
 - 3 To change program, press the multi-function button for 1–2 seconds and release. Repeat until the desired program is reached.
- ! This function is not available while audio streaming with SAMBA 2 GO.




SAMBA 2 GO

7

Streaming with SAMBA 2 GO

Phones, TVs, FM systems—you can connect SAMBA 2 GO to almost any device! SAMBA 2 GO will then wirelessly send the audio from these devices straight to your audio processor.

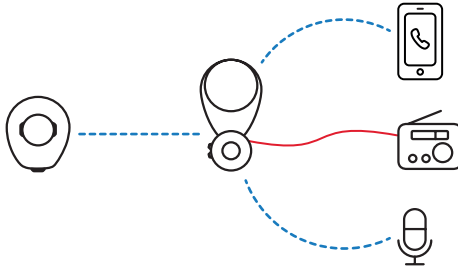

— Cable Connection

- - - Wireless Connection

Audio Streaming Functions

These functions can be used while streaming from any device.

- 1 To start streaming from a Bluetooth device, start audio playback on that device.
- 2 To start streaming from a device connected by cable or an FM receiver, plug the device into SAMBA 2 GO.
- 3 To stop streaming from any device or FM receiver, press the multi-function button for 1–2 seconds until the blue LED turns off.



- 4 To change volume, press the upper or lower volume button.
- 5 To mute your audio processor microphones, press the upper and lower volume buttons simultaneously for 1 second. You will now only hear sound from your audio device.



Pairing with your Bluetooth Device

SAMBA 2 GO can be paired with Bluetooth-enabled phones, TVs, tablets, and many other devices. Every Bluetooth device has to be paired once with SAMBA 2 GO. After that, the connection is automatically established when SAMBA 2 GO is switched on and the Bluetooth device is within operating distance.




- 1 Switch on SAMBA 2 GO and your Bluetooth device.



- 2 Press the multi-function button and the upper volume button simultaneously until the blue LED flashes. This takes approximately 5 seconds.

- 3 Make sure Bluetooth is activated on your device. Search for other Bluetooth devices and select SAMBA 2 GO.
- 4 If requested, enter the PIN "0000".
- 5 Pairing is complete when the blue LED stops flashing, is illuminated for 5 seconds, and then is no longer illuminated.

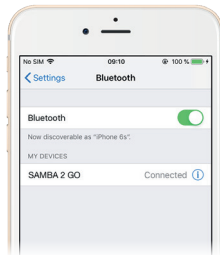

Making Phone Calls

You can use SAMBA 2 GO to make hands-free phone calls—the SAMBA 2 GO microphone will transmit your voice to the other person or people in the call.

- 1 Make sure SAMBA 2 GO is paired with your phone or other Bluetooth device.



- ! If the audio does not stream automatically, disconnect all other audio devices from SAMBA 2 GO. Then press the upper and lower volume buttons together briefly to establish the connection manually.

- 2 To make a call, select the number or contact on your phone as normal. The audio from your phone will automatically be streamed to your audio processor.
- 3 If you receive a call, the ring tone will be streamed to your audio processor and the blue LED will flash. Briefly press the multi-function button to accept the call.
- 4 To reject the call, press the multi-function button for more than 2 seconds.



- 5 To hang up on an active phone call, briefly press the multi-function button.
- 6 To stop streaming the phone call through SAMBA 2 GO, double-press the multi-function button. The phone call will now be heard through your phone's speakers. Double-press the multi-function button to start streaming through SAMBA 2 GO again.



! You can also use the accept and reject buttons on your phone to start or end calls.

Connecting via Cable

You can stream sound from any device with a headphone jack, such as a music player.

- 1 Plug an audio cable into the headphone jack of your device.
- 2 Plug the other end into the audio port of SAMBA 2 GO.
- 3 Select the music you would like to listen to, and press play.
- 4 The music will start streaming to your audio processor.



Using Remote Microphones

SAMBA 2 GO can be connected to FM systems or remote microphones, so you can hear more clearly in challenging environments.



1 Plug the 3-pin receiver into the FM port of the SAMBA 2 GO.

2 Switch on the remote microphone or FM transmitter, according to the manufacturer's instructions.

Charging SAMBA 2 GO

If SAMBA 2 GO has a flashing orange LED light, its battery is low and needs to be charged.



- 1 Plug the USB cable into the USB port.
 - 2 Plug the other end into the power adapter, and then into a wall socket.
 - 3 The orange LED will be illuminated during charging. When the green LED turns on, SAMBA 2 GO is fully charged. This takes around 2 hours.
- ! Only use the USB cable and power adapter that came with the device.



MED-EL Elektromedizinische Geräte GmbH
Fürstenweg 77a, 6020 Innsbruck, Austria
office@medel.com

[medel.com](https://www.medel.com)

